

START AND SHUTDOWN PROCEDURES FOR VIA 400

Thank you for purchasing the new Via 400 by AlterG. To improve your user experience, we have detailed a general start and shutdown procedure for the Via 400 each day. We understand small bugs and hiccups may arise from time to time, however, these procedures ensure you have a great user experience.



START OF THE DAY

Before powering on the Via 400, ensure the cockpit is locked and at the highest position. The software performs a weight calibration on each startup and we want to minimize weight detected to reduce pressure errors.

Press the On/Off switch located on the front cover and turn On the Via 400.

The Via 400 app, aka Buffalo, will auto start within 30 seconds.



END OF THE DAY

Turning off the Via 400 each day will ensure the system resets the software and firmware to optimum efficiency.

Press the PC shutoff button located on the cockpit, next to the Emergency Stop Magnet. A windows screen will appear and will take approximately 30 to 60 seconds to power down, more if there are any windows updates.

Once the PC shuts off, press the On/Off switch on the front cover to the OFF position.



SportsMed Products Limited

www.sports-medical.co.uk
0808 9012015

Unit A4 Brunel Gate, Telford Close, Aylesbury,
Buckinghamshire, HP19 8AR





WE HELP PEOPLE TO GET MOVING

SportsMed Products Ltd is dedicated to getting people moving. We are constantly researching what is new in the market and always looking to uncover new technologies that will help people to recover from injuries or to perform better.

Exclusive distributors of AlterG, as well as many other innovative products to improve fitness, health and recovery, we bring an outstanding service for our customers.

SportsMed Products Limited

www.sports-medical.co.uk
0808 9012015

Unit A4 Brunel Gate, Telford Close, Aylesbury,
Buckinghamshire, HP19 8AR

